



FURIX

Furix Apnea Sensor

**Veterinary Rechargeable
Apnea Sensor**



When the device is connected to the USB charging, it first enters the start-up mode, the red and green lights flash and make a "drip" sound, and then enters the charging mode the red light is on when charging, and the green light is on when the battery is full

Easy Operation

Simplifies monitoring by connecting directly to the pet's tracheal tube and anesthesia machine.

Responsive Design

Alerts with sound and light if the pet stops breathing for 15 seconds, ensuring timely intervention with auto-resume monitoring within 15 seconds.

Smart Features

Touch-activated self-start mechanism and automatic shutdown after inactivity provide hassle-free usage.



Connector

Connect the device and the mask

Hygiene Friendly

Easily cleaned with alcohol wipes for safe and effective maintenance.

Rechargeable Convenience

Equipped with a USB charging system, featuring intuitive red and green indicators for battery status.

Before Use

- **Ensure Proper Connection:** Confirm that the device is securely connected to the pet's tracheal tube and the anesthesia machine to avoid disruptions.
- **Fully Charge the Device:** Charge the sensor using the provided USB cable until the green light indicates a full battery for uninterrupted operation.
- **Check the Sensor's Functionality:** Test the sound and light alarms to ensure they are working correctly before attaching them to the pet.
- **Read the Instructions:** Familiarize yourself with the touch activation, alarm settings, and automatic features for seamless use during procedures.



01 Easy operation

- Just connect the pet choking alarm to the tracheal tube and anesthesia machine

02 Responsive

- Once the pet stops breathing for 15 seconds, the pet choking alarm will sound an alarm. Automatically wakes up the device
- to continue monitoring if it breathes again within a 15-second period

03 Self-start

- Once connected to your pet, touch your finger on the top button to activate the device:

04 Automatic closing

- After disconnecting from the pet or pet stops breathing,
- the pet choking alarm is suspended for 15 seconds,
- and then the alarm sound is automatically turned off after 20

04 seconds Easy to clean

- Wipe with alcohol after use and dry with a dry cotton cloth,
- not soak or sterilize with an autoclave:



Rechargeable



Infrared Touch
Screen Switch



Sound and light
double alarms

Furix Trade Venture Limited

Address: 6/F MEGA CUBE, WANG KWONG ROAD, KOWLOON, HONG KONG,
Mob: +852 9579 0472 E-mail: vet.furix@gmail.com, sales@myfurix.com
Tel: +852 9579 0472

FURIX